

Wheelchair Lap Trays

Reorder #	Description	Wheelchair Size	Unit
705010	Econo -Tray (no padding)	16" - 18"	Each
705018	SofTop Tray w/LSII Cover	16" - 18"	Each
705019	SofTop Tray w/Nylon Cover	16" - 18"	Each
705020	SofTop Tray w/Vinyl Cover	16" - 18"	Each
705021	SofTop Tray w/Vinyl Cover	Fits 14" W/C	Each
705022	SofTop Tray w/Vinyl Cover	20" - 22"	Each
705027	SofTop Tray w/LSII Cover	20" - 22"	Each
705024	SofTop Tray w/Nylon Cover (Bariatric)	24" - 28"	Each
705030	Dura Tray - Plastic	16" - 18"	Each
705091	Strap Replacement Kit	All Trays	Set



Purpose

Wheelchair lap trays are designed to serve as therapeutic interventions that assist patients in achieving proper body position, balance and alignment. Although most residents can remove trays, there are some who can- not. For them the trays could be considered as restraints. Residents who cannot release standard wheelchair lap trays might benefit from Lift-Away Tray (models numbers 705015-705025). These are described in the catalog.

Instructions

1. Place tray on arms of wheelchair arms. Wrap Velcro™ bands around chair arms and tray. Press the hook material against the loop material to secure.
2. The around-the-back strap is intended to prevent the tray from sliding forward when the resident leans against it. Fasten the strap by bringing it around the back of the wheelchair and passing the end through the buckle. Adjust strap to desired length and secure buckle.

Note: SofTop Trays are reversible. The unpadded side may be used when a firm surface is required.

Easy Care

All trays wipe clean using a damp cloth or sponge. Replacement Velcro™ bands (model # 705091) are available from your dealer.